Nasenspülung

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Nose Clean EN

Nasal irrigator

Read the instructions carefully before using this device.

Dear Customer, this nasal irrigator has been designed to treat the upper airways. It can be used with all compressed air aerosols and it nebulizes all aerosoltherapy medicines. If you have any questions or problems or if you need to order spare parts, please contact your dealer or the COLPHARMA customer care service. Alternatively, you can visit www.colpharma.com where you will find lots of useful information about our products.

PREPARATION AND USAGE OF THIS DEVICE

When using the nasal irrigator for the first time, make sure to clean all parts as described in the section "Cleaning and Disinfection".

1. Open the nasal irrigator and take out the nebulizing system (2), and with the chamber to collect the remainder (PIC. 1/A).
2. Pour in the solution or medicine, as prescribed by the physician (PIC. 1/A).
3. Insert the nebulization system, taking care to slot it correctly into the slot on the nebuliser body (PIC. 1/A).
4. Close the nasal irrigator and use the most suitable adapter (ADULT or CHILD). Never force closure and make sure that the slot at the bottom of the adaptor is level with the chamber to collect residues (PIC. 1/A).
5. Connect the nasal irrigator to the compressor air tube (PIC. 1/B).
6. Press the ON/OFF button to switch on the compressor and place the adaptor in the nostril

to be treated. Press the start button for the nebulizer (7) and breathe in.

7. Release the button and gently blow into the nasal irrigator through the treated nostril. Any fluid or mucus residues will be collected in the special chamber.
8. Sit in a relaxed position with the upper body straight. Do not lie down while inhaling.
9. Empty out the remaining medicine from the nebulizer and clean all the parts, as described in the section "Cleaning and Disinfection".

CLEANING AND DISINFECTION

Carefully clean all parts and remove any medicine residues and possible impurities after each treatment. Always wash hands thoroughly before cleaning and disinfecting the accessories.

Wash/sterilise every part of the nasal irrigator.

Cleaning with water

- Clean all the nasal irrigator's parts under hot running water for about 5 minutes, adding a small amount of mild detergent if necessary, following the dosage and limits specified by the detergent manufacturer.
- Rinse thoroughly, taking care to remove all residues, then leave to dry.

Disinfection

- All parts can be disinfected with chemical disinfectants, using doses and limits as specified by the manufacturer of the disinfectant.
- Disinfectants are generally available from pharmacies.

Steam sterilization

- All nasal irrigator parts can be sterilized using steam of up to 100 °C (15-20 min.)
- After sterilizing, always leave the components to cool down to ambient temperature before using again.

Do not repeat the sterilization cycle when the components are still hot.

Sterilization in boiling water

- All of the components can be immersed in boiling water for no longer than 1 minute.
- Boil the components in plenty of water.

Disassemble all components of the nasal irrigator and boil them separately.

MALFUNCTIONS AND WHAT TO DO

- Make sure that the compressor is working.
- Make sure that the air tube is correctly connected at both ends (PIC. 1/B).
- Make sure the air tube is not squashed, bent, soiled or blocked. If necessary, replace with a new one.
- Make sure the nasal irrigator is completely assembled and ensure the components are correctly assembled.
- Make sure the medicine has been poured into the nebulizer.

TECHNICAL DATA

Max filling volume: 10 ml
Min filling volume: 2 ml
Residual volume: 0,6 ml

GUARANTEE

- Spare parts, such as the nose irrigator and its component parts are guaranteed for 1 year against defects in material and workmanship, but not against deterioration caused by normal wear and tear.
- The guarantee does not cover damage caused by improper use, accidents, or failure to abide by the instructions for use.

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