COLPHARMA

Air 10

T H E R A P Y

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Compressed-air aerosol therapy and nasal irrigator system

- | | | | |
|---|-----------------------|----|---|
| 1 | Compressor | 10 | NOSE CLEAN nasal irrigator |
| 2 | ON/OFF button | 11 | Air tube |
| 3 | Air tube outlet | 12 | 5 Air filters |
| 4 | Nebulizer holder | 13 | Components and assembly of EASY 2 PRO nebulizer |
| 5 | Power cable | 14 | Components and assembly of NOSE CLEAN nasal irrigator |
| 6 | Adult mask | 15 | Air filter replacement |
| 7 | Pediatric mask | | |
| 8 | Mouthpiece with valve | | |
| 9 | EASY 2 PRO Nebulizer | | |

Read the instructions carefully before using this device.

BF type part applied.

Dear Customer,

This device is a high-quality aerosol therapy system, ideal for intensive use and chronic illnesses. It is equipped with the innovative EASY 2 PRO nebulizer, which includes an innovative breath-synchronized valve system and the NOSE CLEAN nasal irrigator for treatment of the upper airways. It can be used with the most common aerosol-therapy medicines. If you have any questions or problems or if you need to order spare parts, please contact your dealer or the COLPHARMA customer care service.

Alternatively, visit the www.colpharma.com website where you can find additional information on all our products.

INTENDED USE

This product has been designed for the inhalation of medicines for the treatment of respiratory illnesses.

Intended audience

- Certified medical professionals (physicians, nurses and therapists).
- Patients in their home environment with home health-care support under the directions of a qualified physician.

The user must understand the general operation of the device and the content of the instruction manual.

It is forbidden to use the device on patients who are unconscious or who cannot breathe on their own.

Intended patients

People who suffer from respiratory illnesses and who need to take inhalation medicines.
Suitable both for pediatric and adult use.

Environment

This product has been designed for home use, as well as for medical clinic settings and hospital use.

Minimum product lifespan

The minimum product lifespan periods are reported below and are based on the assumption that the product is used to nebulize a saline solution (2 ml) twice a day, 7.5 minutes at a time, at an ambient temperature of 23 °C.

The lifespan period may vary depending on the usage environment.

- Compressor (main unit): 5 years.
- Nebulizer - nasal irrigator - air tube - mouthpiece - pediatric mask - adult mask: 1 year.
Frequent product use may shorten the expected lifespan.
- Air filter: 60 days.

Warnings

Follow the warnings and precautions reported in the instruction manual.

IMPORTANT PRECAUTIONS

- Follow the instructions for use. This manual contains important information on the operation and safety of this device. Please, read the instructions carefully before using the device and keep them handy for future reference.
- This device must be used exclusively as described in this manual. The manufacturer shall not be liable for any damages caused by improper use.
- The device is not suitable for anesthesia and pulmonary ventilation.
- The device must be used only with its original accessories as listed in the user manual.
- Do not use the device if you deem it damaged or if you notice any anomaly.
- Never open the device.
- This device has been manufactured using sensitive components and it must be handled with care. Please, follow the storage and operation instructions detailed in the “Technical Specifications” chapter.
- Make sure to safeguard the product from:
 - water and humidity
 - extreme temperature
 - impacts and falls
 - contamination and dust
 - direct sunlight
 - heat and cold
- Use this device only with the medicines prescribed by the physician and follow dosing, duration and frequency instructions for the prescribed therapy.
- Do not bend the nebulizer beyond 45°.
- Do not use the device close to strong magnetic fields such as cellular phones and radio installations. Keep a minimum distance of 3.3 m from other equipment when using this device.
- Store the device and its accessories in a clean place.
- Do not use and store this device if smoke or gas is present.
- Do not use any medicines left in the nebulizer. Always use a fresh new dose.
- Do not use this product on anesthesia or pulmonary ventilation circuits.

Children must always use this device under adult supervision. Some parts are small and may represent a choking hazard if swallowed. Cables and tubes may represent a choking hazard.

Electronic devices must be disposed of according to local regulations and not as ordinary household waste.

The use of this device does not replace the medical advice of a physician.

HOW TO PREPARE AND USE THE DEVICE

When using the device for the first time, make sure to clean all parts as described in the section “Cleaning and Disinfection”.

1. Choose whether to use the EASY 2 PRO nebulizer or the NOSE CLEAN nasal irrigator.
2. **EASY 2 PRO nebulizer** (Pic. 2/A):
 - a. Open the nebulizer (9) and pour in the amount of medicine as prescribed by the physician. Make sure not to pass the maximum level mark.
 - b. Choose the most suitable inhalation accessory, as prescribed by the physician. The breath-synchronized valve system maximizes the effect of the medicine, and works only with the mouthpiece (Pic. 2/B).
 - c. When using the mask or to shorten the treatment time with the mouthpiece, open the cap with the inlet valve. The nebulizer will work with continued flow and therefore, more quickly (Pic. 2/C).
 - d. Connect the nebulizer to the compressor using the air tube (Pic. 2/E) and plug in the power cord in an electric outlet making sure that the supply voltage is in compliance with what is stated in this manual or the device nameplate.
 - e. Press the ON/OFF (2) button to turn on the device. Place the mouthpiece (8) in the mouth, rest the mask (6 or 7) against the face, making sure that it covers nose and mouth.
3. **NOSE CLEAN nasal irrigator** (Pic. 2/D):
 - a. Open the nasal irrigator (10) and take out the nebulizing system along with the chamber to collect the residues.
 - b. Pour in the solution or medicine, as prescribed by the physician.
 - c. Insert the nebulization system, taking care to slot it correctly into the slot on the nebulizer body.
 - d. Close the nasal irrigator and use the most suited adult or pediatric adapter.
Never force closure and make sure that the slot at the bottom of the adaptor is level with the chamber to collect residues.
 - e. Connect the nasal irrigator to the compressor air tube (Pic. 2/E) and plug in the power cord in an electric outlet making sure that the supply voltage is in compliance with what is stated in this manual or the device nameplate.
 - f. Press the ON/OFF button to switch on the compressor and place the adaptor in the nostril to be treated. Press the start button for the nebulizer and breathe in.
 - g. Release the button and gently blow into the nasal irrigator through the treated nostril. Any fluid or mucus residues will be collected in the special chamber.
4. Sit in a relaxed position with the upper body straight.
Do not lie down while inhaling.
5. After having completed the therapy session as recommended by the physician, press the ON/OFF (2) button to turn off the device and unplug the power cord from the electric outlet.
6. Make sure that the air tube contains no condensation. In case of condensation, switch on the compressor to dry up any moisture in the tube and then disconnect it as described above.

7. Empty out the remaining medicine from the nebulizer and clean the device in all of its parts, as described in the section “Cleaning and Disinfection”.

 This device has been designed to operate continuously.

 Do not cover the device with a cloth, towel or anything else when in operation to avoid overheating or malfunction of the device.

CLEANING AND DISINFECTION

Carefully clean all parts and remove any medicine residues and possible impurities after each treatment. The compressor (1) and the air tube (11) must be wiped clean with a clean damp cloth.

 Always wash hands thoroughly before cleaning and disinfecting the accessories.

 Wash/sterilize every part of the nebulizer.

 Do not expose the compressor to water or heat.

 Replace the air tube when passing to a new patient or in case of impurities.

 **Before cleaning, disconnect the plug from the electric outlet.**

 **The compressor is not waterproof and must not get in contact with liquids. In case of contact with liquids, immediately disconnect the power cord, shut off the device and dry it with care. Do not immerse the device in liquids.**

Cleaning with water

- Clean all of the device's parts (**except for the air tube**) under hot running water for about 5 minutes, adding a small amount of neutral detergent if necessary, following the dosage and limits specified by the detergent manufacturer.
- Rinse thoroughly, taking care to remove all residues, then leave to dry.

Disinfection

- All parts (**except the air tube**) can be disinfected with chemical disinfectants, using doses and limits as specified by the manufacturer of the disinfectant.
- Disinfectants are generally available from pharmacies.

Steam sterilisation

- All of the components (**except for the air tube and the masks**) can be steam-sterilized up to a temperature of 100 °C (15-20 min.).
- After sterilising, always leave the components to cool down to ambient temperature before using again.

 Do not repeat the sterilisation cycle when the components are still hot.

Sterilisation in boiling water

- All of the components (**except for the air tube and the masks**) can be immersed in boiling water for no longer than 1 minute.

 Boil the components in plenty of water.

 Disassemble all components of the nebulizer and boil them separately.

MAINTENANCE, STORAGE AND SERVICING

Order all the spare parts from your retailer or contact COLPHARMA customer service (see intro).

- It is recommended to replace the nebulizer, mouthpiece, nasal irrigator and masks after 1 year of use.
- Do not dry the nebulizer in the oven or with a hairdryer or its components may be damaged.
- Always check that the filter is clean and replace it immediately if dirty or after a maximum of 60 days of use. Spare filters are provided with the device.
- To replace the filter (15), remove the filter cover from its seat.
Remove the used air filter and insert the new filter. Reposition the filter cover in its seat.
- Do not use the device without a filter installed. Use only original filters.
- Do not wash the filter. If the filter is wet, replace it. If the filter is damp it may cause the device to stop operating.
- Wash the filter cover regularly (do not boil it) then dry it carefully before repositioning it in its seat.

 Never open the compressor compartment.

- Repairs must be performed exclusively by specialized personnel.

MALFUNCTIONS AND TROUBLESHOOTING

The device does not turn on

- Make sure that the plug is well inside the wall socket.
- Make sure to have pressed the ON/OFF (2) button.

Nebulization is weak or not present

- Make sure that the air tube (11) is correctly connected at both ends.
- Make sure the air tube is not squashed, bent, soiled or blocked. If necessary, replace with a new one.
- Make sure the nebulizer is completely assembled (13) and ensure the components are correctly assembled.
- Make sure the medicine has been poured into the nebulizer.
- Make sure the filter is clean and the filter cover is positioned correctly.

The device is noisy

- Make sure to have inserted the filter correctly in its seat (15).

Do not attempt to repair the device. Contact COLPHARMA customer service.

WARRANTY

This device has a **5-year** warranty from the date of purchase. The warranty is valid only with the original purchase receipt

- The 3-year warranty is valid only for the compressor.
- The spare parts such as nebulizer, nasal irrigator, masks, mouthpiece, air tube and filters have a warranty of one year against defects in materials and workmanship, but not against deterioration caused by normal wear and tear.
- Opening or tampering with the device voids the warranty.
- The warranty does not cover damage caused by improper use, accidents or failure to abide by the instructions for use.

TECHNICAL DATA

Nebulizer:	EASY 2 PRO
Max filling volume:	8 ml
Min filling volume:	2 ml
Residual volume:	0.5 ml
Neb rate 1 min (NaCl 0,9%):	0.60 ml/min
Particle size:	3.45 µm - 69% < 5 µm

Nasal irrigator:	NOSE CLEAN
Max filling volume:	10 ml
Min filling volume:	2 ml
Residual volume:	0.6 ml
Neb rate 1 min (NaCl 0,9%):	1.3 ml/min
Particle size:	> 10 µm

Air tube lenght:	1.5 m
Maximum pressure:	2.5 bar
Compressor air flow:	14 L/min
Operating air flow:	5.5 L/min
Acoustic noise level (Compressor without tube):	55 dBA
Operating cycle:	Continuous operation
Operating conditions:	10-40°C / 50-104 °F; 30-85% maximum relative humidity; 700-1060 hPa atmospheric pressure
Storage and shipping conditions:	-20 - +60 °C / -4 - +140 °F; 10-95% maximum relative humidity; 700-1060 hPa atmospheric pressure
Product Weight:	1,860 g
Dimensions:	168 x 144 x 188 mm
Power supply:	Power plug
Expected service life:	2,000 hours
IP Class:	IP21
Reference to standard:	EN13544-1; EN60601-1; EN60601-2; EN60601-1-6; IEC60601-1-1-11

We reserve the right to apply technical changes.

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Shenzhen Horned Medical Device Co., Ltd.
Shanghai International Holding Corp. GmbH