Nose Clean PRO

Nasendusche

DE

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Nose Clean PRO

Nasal irrigator

Read the instructions carefully before using this device.

Dear Customer, the nasal rinse has been designed to treat the upper airways in adults and children. It can be used with all compressed air aerosols and it nebulises all aerosoltherapy medicines. If you have any questions or problems or if you need to order spare parts, please contact your dealer or the COLPHARMA customer care service. Alternatively, visit the www.colpharma.com website where you can find additional information on all our products.

Nebulisation is achieved by allowing the liquid in the nebulisation tank (4.1) to flow from the nasal adapter (1) (CHILD or ADULT) by way of the pressure applied externally by a compressed air aerosol machine. The nebulised water vapour leads to the following technical "nebulisation speed" rate:
Small nasal adapter (CHILD): > than 2.0 ML/min.
Large nasal adapter (ADULT): > than 3.0 ML/min.

NOSE CLEAN PRO consists of an air duct (tube), connector, nebulisation activation switch, spring, nebulisation tank, residue collection chamber, separator, small nasal adapter (CHILD) and large nasal adapter (ADULT), protective cap, and other parts.

HOW TO PREPARE AND USE THE DEVICE

When using the NOSE CLEAN PRO nasal rinse for the first time, make sure to clean all parts as described in the section "Cleaning and Disinfection".

1. Open the nasal rinse by removing the separator (2) (FIG. 1/A)
2. Add the solution (max 20 ml) in the nebulisation tank (4.1). (FIG. 1/A)
3. Pour in the solution or medicine (FIG. 1/A) as instructed by your doctor.
4. Add the separator making sure that it fits in

correctly with the right orientation. (FIG. 1/A)

5. Apply the most suited nasal adapter (1) either for adults (ADULT) or for children (CHILD). Never force closure and make sure that the slot at the bottom of the adapter matches with the residue collection chamber (4.2). (FIG. 1/A)
6. Connect the nasal rinse to the compressor air tube (7). (FIG. 1/B)
7. Place the adapter in the nostril to be treated. Press the start button for the nebuliser (5) and breathe in.
8. Release the button and gently blow into the nasal rinse through the treated nostril. Any fluid or mucus residues will be collected in the special chamber (4.2). During application it is recommended to wear protection against dripping.
9. Sit in a relaxed position with the upper body straight. Do not lie down while inhaling.
10. If necessary, repeat the steps 8 and 9 for the other nostril trying to distribute the quantity of solution evenly between the two nostrils.
11. Empty out the remaining medicine from the nasal rinse and clean all of the parts, as described in the section "Cleaning and Disinfection".

CLEANING AND DISINFECTION

Carefully clean all parts and remove any medicine residues and possible impurities after each treatment. Always wash hands thoroughly before cleaning and disinfecting the accessories. Wash/sterilize every part of the nasal rinse.

Cleaning with water

- Clean all of the nasal rinse's parts under hot running water for about 5 minutes, adding a small amount of neutral detergent if necessary, following the dosage and limits specified by the detergent manufacturer.
- Rinse thoroughly, taking care to remove all residues, then leave to dry.

Disinfection

- All parts can be disinfected with chemical disinfectants, using doses and limits as specified by the manufacturer of the disinfectant.
- Disinfectants are generally available from pharmacies.

Steam sterilisation

- All of the components of the nasal rinse can be steam-sterilized up to a temperature of 100 °C (15-

20 min.).

- After sterilising, always leave the components to cool down to ambient temperature before using again. Do not repeat the sterilisation cycle when the components are still hot.

Sterilisation in boiling water

- All of the components can be immersed in boiling water for no longer than 1 minute.

Boil the components in plenty of water. Disassemble all components of the nasal rinse and boil them separately.

MALFUNCTIONS AND WHAT TO DO

- Make sure that the compressor is working.
- Make sure that the air tube is correctly connected at both ends. (FIG. 1/B)
- Make sure the air tube is not squashed, bent, soiled or blocked. If necessary, replace with a new one.
- Make sure the nasal rinse is completely assembled and ensure its components are correctly assembled.
- Make sure the medicine has been poured into the nebulisation tank (4.1).

TECHNICAL SPECIFICATIONS

Maximum filling volume: 20 ml
Minimum filling volume: 2 ml
Residual volume: 0.6 ml

WARRANTY

- The nasal rinse and its component parts come with a 1-year warranty against defects in material and workmanship, but not against deterioration caused by normal wear and tear.
- The warranty does not cover damage caused by improper use, accidents or failure to abide by the instructions for use.