MAX
PROTECTION

Just Warm It

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Just Warm It

EN

After-bite thermal treatment against insect bites and stings

1. Device status LED
2. ON/OFF button
3. Start button
4. Ceramic heating plate
5. Battery compartment
6. Silicone cord

Read the instructions carefully before using this device.

Just Warm It thermal after-bite is an innovative medical device designed to effectively alleviate itching and swelling caused by insect bites or stings through the application of heat (localized hyperthermia).

OVERVIEW

After experiencing an insect bite, such as a mosquito, bee, bed bug, tick, or spider, the human body initiates an immune response, releasing histamine. Histamine is a chemical substance with a key role in allergic and inflammatory reactions.

The release of histamine binds to nerve cell receptors, leading to the dilation of blood vessels and triggering common symptoms such as swelling, itching, redness, and pain in the vicinity of the bite or sting. Clinical studies have demonstrated that the targeted application of heat immediately

following an insect bite or sting positively impacts the immune response, reducing the release of histamine and inflammatory agents, providing prompt relief from pain and swelling.

Additionally, heat fights itching by activating TRPV-1 receptors, which inhibit the neural pathways that convey this sensation. The application of heat serves as a proactive “counterstimulus” on the skin’s nerve cells, effectively blocking the transmission of the itching signal to the brain and diminishing its perception.

Clinical findings demonstrated that heat treatment provides swifter relief from itching, thereby reducing its duration compared to untreated cases.

SIDE EFFECTS

Do not use the device:

- If you suffer from diabetes. Pain sensitivity may be reduced in certain circumstances increasing the risk of burns.
- On individuals with disabilities, children under 4 years of age, or those with diminished sensitivity to heat. It also includes people with skin alterations due to diseases or scarred tissues in the application area, after taking pain medications, consuming alcohol, or using drugs.
- On the skin affected by acute or chronic conditions (wounds or irritation), such as inflamed, reddened, rash-prone, allergic, burned, frozen, bruised, or swollen skin (without signs of insect bites), open or healing wounds, and surgical scars where the healing process may be compro-

mised.

- If running a fever.
- In case of disorders reducing the sensation of pain (e.g., metabolic disorders).
- In case of use of lotions, creams, and gels applied locally.
- In case of skin irritation due to prolonged heat application.

SAFETY

Before using this device, please carefully review the user's instructions, specifically those pertaining to safety, and store them for future reference. In the event that the device is transferred to another person, it is imperative to include in the transfer these instructions.

WARNINGS

- You must be 12 years old or older to use this device alone. Use on children under 12 years of age must occur under adult supervision or with the assistance of an adult.
- Before using the device, consult a doctor in case of skin conditions, serious illnesses (especially if there is a predisposition to thromboembolic conditions or recurrent malignant growths), or unexplained chronic pain in any part of the body.
- Consult a doctor immediately if symptoms worsen during or after the

use of the product and discontinue use.

- Seek immediate medical attention if dealing with a tick bite. Tick bites can transmit pathogens such as tick-borne encephalitis (TBE) or Lyme disease.
- Seek medical attention immediately if the first signs of increased itching, skin redness, swelling, dizziness, nausea, shortness of breath, rash formation, or a decrease or increase in blood pressure occur.
- Do not use near the eyes, on the eyelids, on the mouth, or on mucous membranes. Additionally, do not use the device on wet skin.
- Do not allow children to play with the device.
- Keep the device out of the reach of children and pets.
- Keep the packaging material away from children (risk of choking).
- Observe general hygiene measures when using the product against insect bites.
- The device does not replace medical consultation and treatment. Always consult your doctor in case of pain or illness.
- If the device is not working correctly or if discomfort or pain is experienced, discontinue use immediately.

GENERAL PRECAUTIONS

- Do not touch the device with wet hands. Ensure that water does not enter the device. If this happens, remove the batteries and use the device again only when it is completely dry.
- Do not use detergents or solvents for cleaning and maintaining the device. The penetration of liquids into the device may damage the elec-

tronics or other parts of the device and cause a malfunction.

- Protect the device from direct sunlight and store it in a dry place.
- Shield the device from shocks, moisture, dust, chemicals, drastic temperature fluctuations, and heat sources (ovens, stoves).
- If you store the device, do not place heavy objects on it.
- Before use, ensure that the device has no visible damage. In case of doubt, do not use the device and contact the retailer or customer service.

INSERTING AND REPLACING THE BATTERIES

1. Open the battery compartment cover (5).
2. Remove the exhausted batteries (if present).
3. Insert the batteries into the battery compartment (5) as indicated in the compartment. When inserting the batteries, make sure to match the correct polarity (+/-).
4. Slide the battery compartment cover (5) until it clicks into place.

Remove the batteries if you do not intend to use the device for an extended period (more than 3 months).

USE

WARNING

Before use, make sure that no insect stingers are present in the area where you have been stung. Remove the stinger carefully before using the device. Immediate use after the insect bite or sting ensures better results. If you wait too long to treat the bite or sting, and itching and swelling start, the thermal after-bite Just Warm It may have a limited effect.

1. Press the ON/OFF (2) button to turn on the device. The device status LED (1) turns green, and an audible signal indicates that the device is ready for use.
2. For a 3-second application, place the device heating ceramic plate (4) on the area where the sting or bite has occurred. Press the activation button (3) once. The device status LED (1) turns blue and an audible signal indicates that the device is starting the application. **IMPORTANT:** The 3-second application begins once the temperature of $47\text{ }^{\circ}\text{C} \pm 2\text{ }^{\circ}\text{C}$ ($116.6\text{ }^{\circ}\text{F} \pm 3.6\text{ }^{\circ}\text{F}$) is reached. This short application time is recommended for the initial use, for use on sensitive skin, and for use on children (4 years of age and older).
3. For a 6-second application, place the device heating ceramic plate (4) on the area where the sting or bite has occurred. Press the activation button (3) twice in quick succession. The device status LED (1) turns orange and an audible signal indicates the start of the application. **IMPORTANT:** The 6-second application begins once the temperature of $50\text{ }^{\circ}\text{C} \pm 2\text{ }^{\circ}\text{C}$ ($122\text{ }^{\circ}\text{F} \pm 3.6\text{ }^{\circ}\text{F}$) is reached. This application time is recommended for regular use on normal skin and adults.

ATTENTION

- Depending on the sensitivity of the person using the device and the skin area, the treatment temperature may be uncomfortable and cause slight skin redness, and in rare cases, irritation. If the 6-second function for normal skin feels too hot, try the 3-second function for sensitive skin and children or discontinue the treatment.
- Use Just Warm It on any other bites/stings after a 2-minute break. Do not exceed a maximum of 5 applications per hour on the same area treated.
- If the device is no longer needed, press the ON/OFF button (2) to turn it off. The green status LED of the device (1) turns off. If no further applications are made within 5 minutes, the device turns off automatically.
- Before storing it, make sure the device is turned off, and the ceramic plate (4) has cooled down.

Battery Warning

Before the battery charge level becomes too low for further applications, the device status LED (1) starts flashing three times, and an audible signal is emitted three times. Replace both batteries.

Issue	Possible cause	Solution
The device does not turn on.	Installation of the batteries is incorrect.	Reinstall the batteries.
	The batteries are depleted.	Replace both batteries as described in these user's instructions.
The device is no longer heating properly.	The battery charge level is too low.	Replace both batteries as described in these user's instructions.
The device emits 3 audible signals/the device status LED (1) flashes, even after turning the device off and on.	The battery charge level is too low; the device may no longer be able to operate.	Replace both batteries as described in these user's instructions.
The application cannot be started, even if the device status LED (1) turns green.	The device needs to be turned on again.	Press the ON/OFF button again to turn on the device.

DEVICE CARE AND CLEANING

- It is advisable to clean the device before the first use and after each use.
- The device must be turned off, allowed to cool, and the batteries must be removed each time before cleaning.
- For cleaning, use only a soft and dry cloth.
- Never use detergents that may be toxic if they come into contact with the skin or mucous membranes, ingested, or inhaled. Do not use abrasive products for cleaning, and never immerse the device in water.
- This device is not dishwasher safe.
- Avoid cleaning with brushes or similar tools, which may cause irreparable damage.
- When using the device on multiple stings or bites, it is advisable to clean the ceramic heating plate (4) every time. This reduces the possibility of transferring germs and bacteria from one area of the skin to another. To disinfect the ceramic heating plate (4), use commercially available alcohol disinfectant wipes.

DISPOSAL

This product must not be disposed of with household waste. All users must deliver electrical or electronic devices, regardless of whether they contain toxic substances or not, to a municipal or commercial collection point, so they can be disposed of in an environmentally friendly manner. Remove the batteries before disposing of the device/unit. Do not dispose of old bat-

teries with household waste, but take them to a battery collection station at a recycling site or a store. For information on disposal, refer to the local authorities or the supplier.

TECHNICAL SPECIFICATIONS

Type:	Thermal after-bite
Model:	IBR10
Dimensions:	94.5 × 34 × 20 mm
Weight (with no batteries):	25 g
Power supply:	2 batteries, type AAA (LR03 1.5V)
Maximum temperature:	50 °C ± 2 °C (122 °F ± 3.6 °F)
Functions:	Application for sensitive skin/children: 3 seconds, 47 °C ± 2 °C (116.6 °F ± 3.6 °F) Application for normal skin/adults: 6 seconds, 50 °C ± 2 °C (122 °F ± 3.6 °F)
Useful life:	5 years
Duration:	Approximately 300 applications (per battery) expected.
Storage and transportation requirements:	From -10 °C to +50 °C, relative humidity from 30% to 85%, pressure from 700hPa to 1060hPa
Operating environment requirements:	From 10 °C to 40 °C, relative humidity from 30% to 85%, pressure from 700 hPa to 1060 hPa.

We reserve the right to apply technical changes

WARRANTY

This device is covered by a 2-year warranty from the date of purchase, against manufacturing defects, valid only upon submission of the purchase receipt.

The warranty does not include:

- costs and risks associated with transportation;
- damage due to improper use or failure to follow the operating instructions;
- damage due to falls;
- packaging/storage materials and operating instructions.

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AOJ HEALTH Technology Co.,Ltd
Share Info GmbH