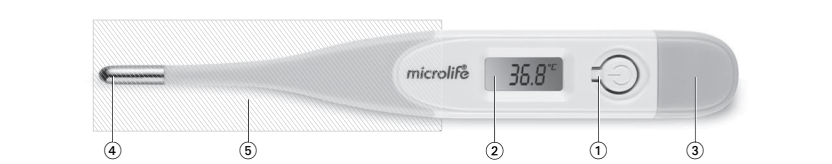
MT 810

IB MT 810 IT-V4 3425, Revision Date: 2025-09-08

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Description of this thermometer	EN
1 ON/OFF button 2 Display 3 Battery compartment cover 4 Measuring sensor 5 Cleaning and disinfecting area (thermometer probe only)	
Device installation and setup A battery is pre-installed in the device. When the low battery symbol «⚡» appears in the display, the battery is flat and needs replacing. Remove the battery compartment cover (3) from the thermometer by using proper instrument. Replace the new battery with the «+» at the top. NOTE: Make sure you have a new battery on hand. For the correct battery type, please refer to the specifications. Keep the battery away from children/infants. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.	
Important information Device description A digital thermometer is a medical device that utilizes the principles of conductive thermal equivalent between device-body contact and digital signal processing to compute and provide a body temperature measurement. Intended purposes The device is intended to measure body temperature. Intended user The device is intended to be operated by adults and adolescents with adequate vision, motor functions, and education, capable of understanding the instructions for use and operating general household electrical appliances. Intended patient The intended patients are the general population of all ages. Intended use environment and conditions The device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a care giver, and in a professional healthcare facility (e.g. physician office). Indications This device measures body temperature for indications of: - Screening for illness and conditions related to high or low body temperature. - Do not attempt rectal measurements on persons with rectal disorders. WARNING! NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient. - This device may only be used for the purposes described in this instruction manual. The manufacturer cannot be held liable for damage caused by incorrect use. - The measurement result of this device is not a medical diagnosis and not intended to substitute consultation and diagnosis by a qualified professional healthcare provider (e.g., physician, pharmacist, or other licensed health-care professionals). Do not use this device for self-diagnosis or for self-treatment of a medical condition. Seek advice from a health-care professional immediately if the patient is clearly unwell and/or having physiological or medical symptoms. - This product contains a button cell battery. A swallowed button cell battery can cause internal chemical burns in as little as two hours and lead to death. Dispose of used batteries immediately. Keep new and used batteries away from children/infants. If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention. - Do not attempt to measure temperature of the body sites that are not specified in this instruction. - Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed. - Do not walk or run when taking temperature. - For safety reasons (risk of rectal perforation) rectal measurement in children younger than 3 years must be performed only by trained health care personnel (professional user). Use another measurement method instead. For rectal fever measurement in small children younger than 3 years, thermometers with a flexible probe are available. - Professional users must follow the country specific applicable medical devices ordinance.	


Important information
Device description A digital thermometer is a medical device that utilizes the principles of conductive thermal equivalent between device-body contact and digital signal processing to compute and provide a body temperature measurement. Intended purposes The device is intended to measure body temperature. Intended user The device is intended to be operated by adults and adolescents with adequate vision, motor functions, and education, capable of understanding the instructions for use and operating general household electrical appliances. Intended patient The intended patients are the general population of all ages. Intended use environment and conditions The device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a care giver, and in a professional healthcare facility (e.g. physician office). Indications This device measures body temperature for indications of: - Screening for illness and conditions related to high or low body temperature. - Do not attempt rectal measurements on persons with rectal disorders. Contra-indications - The device is not suitable for use on sites with any breached or compromised skin. - Do not attempt rectal measurements on persons with rectal disorders. WARNING! NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient. - This device may only be used for the purposes described in this instruction manual. The manufacturer cannot be held liable for damage caused by incorrect use. - The measurement result of this device is not a medical diagnosis and not intended to substitute consultation and diagnosis by a qualified professional healthcare provider (e.g., physician, pharmacist, or other licensed health-care professionals). Do not use this device for self-diagnosis or for self-treatment of a medical condition. Seek advice from a health-care professional immediately if the patient is clearly unwell and/or having physiological or medical symptoms. - This product contains a button cell battery. A swallowed button cell battery can cause internal chemical burns in as little as two hours and lead to death. Dispose of used batteries immediately. Keep new and used batteries away from children/infants. If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention. - Do not attempt to measure temperature of the body sites that are not specified in this instruction. - Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed. - Do not walk or run when taking temperature. - For safety reasons (risk of rectal perforation) rectal measurement in children younger than 3 years must be performed only by trained health care personnel (professional user). Use another measurement method instead. For rectal fever measurement in small children younger than 3 years, thermometers with a flexible probe are available. - Professional users must follow the country specific applicable medical devices ordinance.

NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient.

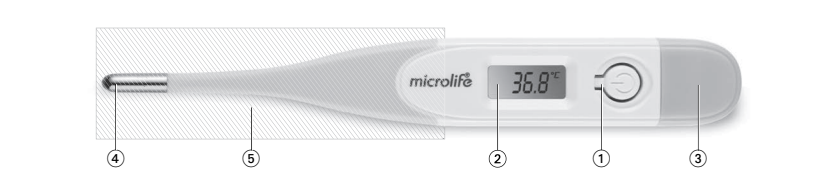- This device may only be used for the purposes described in this instruction manual. The manufacturer cannot be

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